INSTRUCTIONS FOR USE

Albumin Urine Test Home Collection kit

Read these instructions carefully before collecting your sample. Your test results may not be valid if your sample is not collected and packaged properly.

Kit Includes:

- 1 Collection instructions
- 2 Packets containing castile soap towelettes
- 1 BD Vacutainer® urinalysis cup kit containing:
 - 1 Urine collection cup
 - 1 Yellow-top collection tube
- 1 Biohazard bag containing:
 - 1 Absorbent sleeve
- 1 Labcorp eREQ test request form
- 1 Cardboard box (4" x 3" x 1") for sample return
- 1 Self-sealing, postage paid mailing envelope

Preparation

Step 1

Open the urine collection kit and take out all of its contents.

Remove the blue lid from the urine collection cup. Be careful not to touch the inside of the cup. Place the lid on a counter with the tube facing up. Do not touch the inside of the lid or the tube.

Step 2

Open both castile soap packets, but do not remove the towelettes. Wash your hands thoroughly with soap and water.

Step 3

Use the castile soap towelettes to clean your urinary opening and the area around it as follows:

For males: Use the first towelette to clean the tip of your penis, starting at the urinary opening and working your way outward. If you are uncircumcised, you must first pull back the foreskin. Repeat using the second towelette.

For females: Use your fingers to separate the folds of skin around your urinary opening. Clean the urinary opening and the area around it with the first towelette. Repeat using the second towelette.

Collection

Step 4

Collect urine into the collection cup as follows:

- Allow the first part of your urine stream to flow into the toilet.
- As you continue to urinate, place the collection cup below your urinary opening to catch the middle part of your urine stream. Fill about half the cup. Do not touch the inside or rim of the collection cup with your fingers,hands, or any other part of your body.
- Remove the collection cup and let the rest of your urine stream flow into the toilet.

Close the collection cup with the blue lid, touching only the outside of the cup and lid.


Step 5

Peel back, but do not remove, the yellow sticker on the lid of the collection cup.

Insert the yellow-top collection tube topside down into the hole in the lid. Push firmly on the tube until urine starts to flow into the tube. Hold the tube in place until the urine flow stops.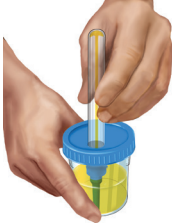

Step 6

Pull the collection tube out of the hole and set it aside. Cover the hole in the collection cup lid with the yellow sticker. Take the lid off the collection cup and pour the remaining urine into the toilet. Put the lid back on the collection cup and throw it away in the trash.

Packaging

Step 7

Place one label from the top of the Labcorp eREQ test request form on the side of the specimen tube. Note: Your specimen will not be tested if this label is missing.

Step 8

Take the absorbent sleeve out of the biohazard bag, and place the yellow-capped collection tube in it. Put the sleeve containing the tube into the biohazard bag, seal the bag, and put the sealed bag in the 4" x 3" x 1" cardboard box. Close the lid of the cardboard box.

Step 9

Complete the test request form by printing your name, date of birth, and the date and time of collection in the spaces provided and signing the bottom of the form.

Step 10

Put the cardboard box containing your specimen and the test request form into the mailing envelope. Remove the plastic strip from the envelope flap to expose the adhesive. Fold over the flap and firmly press along the entire length of the adhesive strip to seal the envelope.

Within 24 hours, place the envelope containing your specimen and test request form in the US mail for direct delivery to the laboratory. Your sample will not be processed unless mailed in the pre-labeled, postage-paid return envelope.

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