Let's get started.

Kit includes:

This Kit includes **two (2)** collections. **Do not open the collection bags until indicated in the steps below.** Please be prepared to ship **Collection 1** and **Collection 2** to the laboratory for processing within 24 hours of collection.

Collection overview

Specimen confirmation form

FedEx envelope (located at the bottom of the kit)

Collection 1

Collection 2

1

You do not need to fast prior to your collections. You can eat and drink as normal.

2

When you are ready to begin the collection process, open only the blue bag marked Collection 1. Follow the instructions inside the blue bag for Collection 1.

3

Once you have completed Collection 1, open only the yellow bag marked "Collection 2" and follow the instructions inside the bag for Collection 2.

4

Once you have completed Collection 2, follow the instructions inside the bag for Collection 2 to prepare both collections for shipment. You will need the specimen confirmation form and the FedEx envelope. Please refer to back of this page for shipping instructions.

Return Shipping Instructions

Your package containing the specimens from both collections is to be mailed back to Labcorp via **FedEx** in the prepaid FedEx mailer included in your kit. **Do not** place your specimen in your USPS mailbox. **It's important to ship your specimen to Labcorp within 24 hours of collection.**

Option 1 FedEx Drop Box

You can drop off in a FedEx drop box.
Visit <https://bit.ly/fedexdropbox>
or **scan QR code** to view FedEx drop box pickup schedules.

Option 2 Schedule a FedEx Pickup

Call FedEx at 800-463-3339 and answer some questions.

FedEx: "What are you calling about?"

You: "FedEx Express return pickup."

FedEx: "...a pickup for today or tomorrow, depending on availability, say 'pickup.'"

You: "Pickup."

FedEx: "Is the word "stamp" written anywhere on your return label?"

You: "No."

FedEx: "What is the tracking number?"

You: Provide the tracking number (TRK#) from the return label.

FedEx: "What kind of return are you calling about?"

You: "FedEx Express." Provide additional information as requested. You may be asked for the pickup location address, number of packages to be picked up, day and time window for the pickup, or if your shipment is currently ready.

Note the confirmation pickup number provided.

If you are asked for a credit card or account number, state that this is a FedEx Express return pickup and provide the tracking number (TRK#) from the return label.

ORIGIN ID: GSXA (423) 423-4234	SHIP DATE: 11FEB21 ACTWGT: 1.00 LB CAD: 114076797/WVSX3200
RONNY ROY 3061 S CHURCH ST BURLINGTON, NC 27215 UNITED STATES US	
TO ATTENTION: LABCORP, DIRECT TEST DEP LABCORP 1447 YORK CT **TEST LABEL - DO NOT SHIP** BURLINGTON NC 27300	
INV: (336) 436-7597	REF: DEPT:
RMA:	
	
RETURNS MON-SAT PRIORITY OVERNIGHT	
27300	
NC-US	
	

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