Collection 2


Includes:

2 Castile soap towelettes

1 Urine collection cup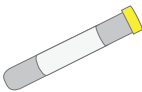

1 Yellow-top collection tube

1 Absorbent sleeve

1 Biohazard specimen bag


1 Cardboard return box

Collection 2 can be collected at any time of the day and does not require fasting. Please be prepared to ship **Collection 1** and **Collection 2** to the laboratory for testing within 24 hours of collection.

1 **Read through the instructions completely before collecting the sample.** Test results may be invalid if sample is not collected properly.

2 **Open the bag marked “Collection 2” and place all the contents on a clean, dry surface.** It may be helpful to place on a counter near or in the bathroom so you can quickly reach it from the toilet.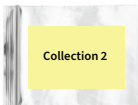

3 **Locate the yellow-top collection tube, and confirm your name is properly printed on the label.** Remove the blue lid from the urine collection cup. Place the lid on a counter with the yellow sticker facing down. Do not touch the inside of the cup or lid.

4 **Open both castile soap packets and place the open packets on a counter, but do not remove the towelettes.** Wash your hands thoroughly with soap and water.

5 **Use the castile soap towelettes to clean your urinary opening and the area around it. Dispose of towelettes in trashcan.**

For Males: Use the first towelette to clean the tip of your penis, starting at the urinary opening and working your way outward. If you are uncircumcised, you must first pull back the foreskin. Repeat using the second towelette.

For Females: Use your fingers to separate the folds of skin around your urinary opening. Clean the urinary opening and the area around it with the first towelette. Repeat using the second towelette.

6 **Collect urine into the collection cup as follows:**


- Allow the first part of your urine stream to flow into the toilet
- As you continue to urinate, place the collection cup below your urinary opening to catch the middle part of your urine stream. Fill about half the cup. Do not touch the inside or rim of the collection cup with your fingers, hands or any other part of your body
- Remove the collection cup and let the rest of your urine stream flow into the toilet

Touching only the outside of the cup and lid, close the collection cup with the blue lid.


7 **Peel back, but do not completely remove, the yellow sticker on the lid of the collection cup.** If the yellow sticker comes off completely, set the sticker aside to use in step 10 below.

8 **Place the unopened yellow-top collection tube topside down into the hole in the lid.** Push firmly on the tube until urine starts to flow into the tube. Hold the tube in place until the urine flow stops. **Do NOT open the yellow tube.**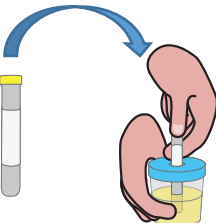


9 **Remove the unopened yellow-top collection tube from the hole and set it aside.**


- Cover the hole in the blue lid with the yellow sticker
- Take the blue lid off and pour the remaining urine into the toilet
- Put the blue lid on the collection cup and throw it away in the trash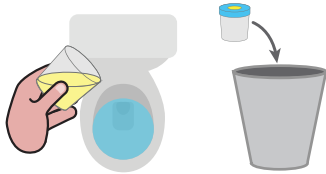

10 **Locate the label of the yellow-top collection tube, and in ink, write today's date.**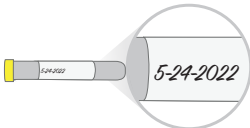

11 **Place the yellow-top collection tube into the absorbent sleeve.**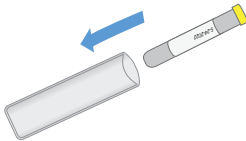

12 Put the sleeve containing the tube into the biohazard bag, seal the bag and place into the cardboard return box. Close the lid.


TO PREPARE COLLECTION 1 AND 2 FOR SHIPMENT:

1 **Fill out the specimen confirmation form.** In ink, include the date, time of collection and your signature in the Patient Section.

2 **Put the white foil pouch from Collection 1, the cardboard box from Collection 2 and the completed specimen confirmation form into the FedEx envelope.** Peel tape from flap of envelope and press flap firmly to seal.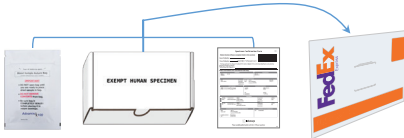

3 **Return the postage paid, pre-addressed FedEx envelope to Labcorp.** You can deliver it to a FedEx drop box or schedule a pickup. Please reference the back of the Kidney Health Kit Overview piece included in your kit for more information. **PLEASE SHIP WITHIN 24 HOURS OF COLLECTION.**

4 **Results are available in the Labcorp Patient™ portal 7 to 10 days after the samples arrive at the lab.** Visit patient.labcorp.com to set up or access your account. To view a dependent's results, add them to your account.

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