Labcorp ColoFIT™ Home Collection Kit

Let's get started.

Kit includes:

Green-cap sample collection device

Collection paper

Absorbent pad

Specimen confirmation form

Biohazard specimen bag

Hard mailing envelope

1 Read through the instructions completely before collecting the sample. Test results may be invalid if sample is not collected properly. Please check the label on the sample collection device and confirm it is your name printed on the label.

2 Open your kit and place all the contents on a clean, dry surface. It may be helpful to place on a counter in the bathroom so you can quickly reach it from the toilet.

3 Place the collection paper inside toilet bowl on top of water. If you need to urinate, do so before placing the collection paper in the toilet to keep the top of the paper dry.

4 Deposit stool sample on top of collection paper.

5 Hold collection device vertically and open green cap by twisting and lifting the probe from the device.

6 Scrape the surface of the fecal sample with the sample probe. Cover the grooved portion completely with stool sample. Collect sample from stool before paper sinks and stool sample touches water.

7 Close collection device by inserting the sample probe and snap green cap on tightly. Do not reopen.

8 Flush. Collection paper is biodegradable and will not harm septic systems.

9 Wrap the collection device in the absorbent pad and insert into the biohazard specimen bag.

10 Fill out the specimen confirmation form. Include the collection date and time and your signature in the Patient Section.

11 Place the sealed biohazard bag containing your specimen and completed specimen confirmation form into the hard mailing envelope.

12 Peel tape from flap of envelope. Fold flap at line. Press flap firmly to seal.

13 Place the prepaid sealed mailing envelope in your mailbox to return to the lab.

Sample must be sent within 24 hours of collection.

14 Results available in the Labcorp Patient™ portal 7 to 10 days after the sample arrives at the lab. Visit patient.labcorp.com to set up or access your account.

15 To view a dependent's results, add them to your Labcorp Patient portal account. Results are also sent to the health care provider who ordered this testing for you.

To see a video, scan QR code:

Or go to labcorp.com/ColofitInstructionsID

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