Litholink 24-hour At-Home Urine Testing

Instructions for use

Labcorp offers the Litholink 24-hour At-Home urine test that is specialized for kidney stone formers. Your provider has requested that you complete a Litholink At-Home kit. Your provider is waiting on these test results in order to start your kidney stone treatment plan.

Verify that you have received all necessary collection materials. Keep the pre-labeled patient sample shipping box; it will be used for returning your samples.

1 collection container per specimen

1 green-topped 50ml tube per specimen

1 tube of liquid urine preservative per specimen

1 biohazard bag with absorbent paper

1 collection aid (Included for females)

Collection data form and insurance information

1 pre-labeled patient sample shipping box

Please watch our instruction video. Scan the QR code or go to litholink.labcorp.com/services/urine-kits-collection and click the blue button "24-hour Urine Collection Instruction Video". This will review step-by-step collection instructions for you.

Important reminders

- If your physician gave you a paper test requisition form for the Litholink test, it must be included in the return shipment with your sample.
- You must collect all of your urine for 24 hours. (We will accept between 22–26 hours.)
- The preservative (the liquid, uncapped bottle, and cap) must be dropped into the large orange jug at the start of your collection.
- Do not let stool contaminate your urine collection.
- Stop taking Vitamin C (pill form, vitamins, and/or supplements) that is greater than 100 mg per day 3 days prior to the start of your At-Home kit. Vitamin C occurring in foods and drinks can be ingested as normal.
- Your sample is extremely time sensitive; you must ship your sample(s) the same or next business day (Mon–Sat). Any sample received after 96 hours (4 days) from the date you ended your collection will be rejected.
- Do not place urine in refrigerator at any time.

For more information, FAQs, or to contact us, visit litholink.labcorp.com or email us at LitholinkInquiry@labcorp.com

Collection instructions

Do not record any information on this sheet, please use the Collection Data form.

Collection 1

HELPFUL HINT

Schedule your FedEx pickup for the day you plan to finish your collection(s). See *Shipping Instructions Insert*.

1


When you wake up in the morning, flush your first urine in the toilet.

This is the START TIME.

2


Record this time on the *Collection Data Form* where it says **START TIME**.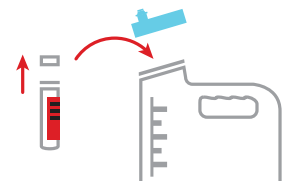

3


Open the tube of urine preservative and empty it into the collection container.

Handle preservative with care and keep out of reach of children.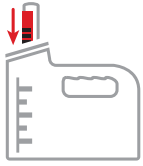

4 Drop the **urine preservative tube and lid** into the collection container. This ensures every drop of the preservative gets into the container.

Important: Do not place urine in refrigerator at any time.

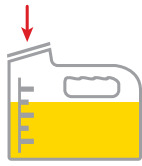
5 Collect all of your urine into the container over the next 24 hours, **including any urine collected during the night and that first urine of the following morning.**

This is the STOP TIME. For women who may have trouble urinating directly into the collection container, place the collection aid over the toilet and then pour the urine into the collection container.

6 Record the **STOP TIME** on the *Collection Data Form* where it says **STOP TIME**.

Record the date you finished the collection on the *Collection Data Form* where it says **DATE COLLECTION ENDED**.

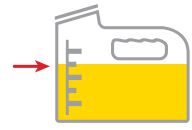
7 Place the collection container on a flat surface and use the measuring tool along the side of the container to read how much fluid is inside the container.


This is the TOTAL VOLUME.

8 Record this **TOTAL VOLUME** on the *Collection Data Form* where it says **TOTAL VOLUME**.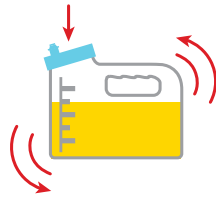

9 Secure lid and shake the collection container.

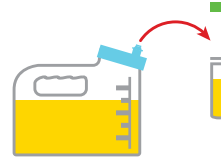10 Fill the green-topped tube marked Collection 1 about 3/4 full with the urine sample.

If you have been instructed to do two 24-hour collections, fill the second green-topped tube marked Collection 2 about 3/4 full from second collection container.

11 **Twist top tightly to seal.**

12 Flush the remaining urine. Do not discard the preservative bottle or cap into the toilet. The jug, preservative, and cap can be discarded into the trash.


If you have been instructed to complete two 24-hour collections, continue with the instructions for **Collection 2, otherwise skip to the shipping instructions.**

1 **The START TIME of the second collection is the same as the STOP TIME of the first collection.**

Record the **START TIME** of the second sample on the *Collection Data Form*.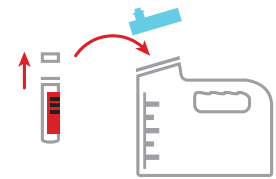

2 Repeat steps from **Collection 1** starting at step 3.

Shipping

Once your collection is completed, put the green-topped sample tube(s) in the biohazard bag in the return box, along with the data/insurance forms, and test requisition (if provided). Please refer to the enclosed *Shipping Instructions* insert to ship your samples via FedEx.

Do not return collection containers.

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©2023 Laboratory Corporation of America® Holdings. All rights reserved. DX_IFU2_365101_0323_1